



## Mirage Activa™ nasal mask

### FITTING GUIDE

#### Fitting the Mirage Activa nasal mask



1

- With one headgear clip released from mask, place mask cushion on nose and pull headgear over the head.
- Connect free headgear clip to mask frame.



2

- Adjust upper and lower headgear straps until mask cushion sits loosely on face, with forehead pads lightly touching forehead.



3

- If forehead pads do not touch forehead, adjust forehead support by depressing tab.
- To move tab up, tilt pads forward. To move tab down, tilt pads backward.



4

- Connect one end of main air tubing to flow generator and other end to mask swivel.
- Connect mask elbow to mask frame—push firmly until it clicks into place.



5

- Turn flow generator on and allow cushion to inflate. If there are air leaks around cushion, lift mask slightly off face and reposition.
- You may need to readjust straps and/or forehead support. Do not overtighten straps—ensure that ActiveCell chamber of mask cushion is moderately and evenly inflated.
- To remove mask, squeeze headgear clips and pull mask and headgear away from head.

